

Medical PM^{MD}

Replacing Gas Hoses

Articulated Arms and Service Managers

Why should you replace the gas hoses?

In order to avoid compromising the safety of patients due to the malfunctioning of the gas hoses present in the articulating arms and the service managers, **the manufacturer recommends replacing the gas hoses before the tenth (10) year of their production anniversary***.

In addition, **manufacturers of gas hoses recommend that the gas hoses be inspected regularly at least once a year**; the inspection protocol should comply with NFPA 99 for gas and suction systems.*.

Replace hoses with Medical PM is:

- Considering the patient's safety and avoiding problems with potentially dramatic consequences
- Guaranteeing the integrity of the equipment in accordance with the manufacturer's standards
- Preventing occlusions
- Determining the length of the gas hoses relative to the type of articulated arm and service manager
- Reducing the risk of bursting or leakage of gas hoses
- Increasing the ease of movement of the articulated arm
- Following the CSA code protocol
- Assuring the expertise of our skilled and experienced technicians
- Sending documentation to customers demonstrating that the equipment has been replaced and gas outlets have been inspected,



What does the technician do when changing the gas hoses?

- Ensures that the Hospital's infection control standards are met through our removable partitions to isolate areas of intervention
- Ensures that there are no obstructions in the gas hoses by checking the gas outlets
- Checks the wear of the electrical wires and the low voltage cabling in the articulated arms
- Replaces all gas hoses in accordance with color codes and DISS connectors
- Examines all gas outlets after hose installation
- Inspects the articulated arms and, if necessary, cleans the joints
- Checks the gas flow of the gas outlets
- Eliminates old gas hoses replaced by leaving the premises clean and functional
- Medical PM prepares a detailed report for your files and advise if any problem is detected



*Documentation is available upon request

For an estimate, contact us

Toll Free: 1 (855) 404-7676

Email: info@medical-pm.com

www.medical-pm.com

